


For shipping purposes, the Therapak<sup>®</sup> Rolling Tote is delivered in a collapsed state. These 5 simple steps will get your Rolling Tote ready to go in seconds!

**Step 1:** Completely unzip the top portion of the Rolling Tote.

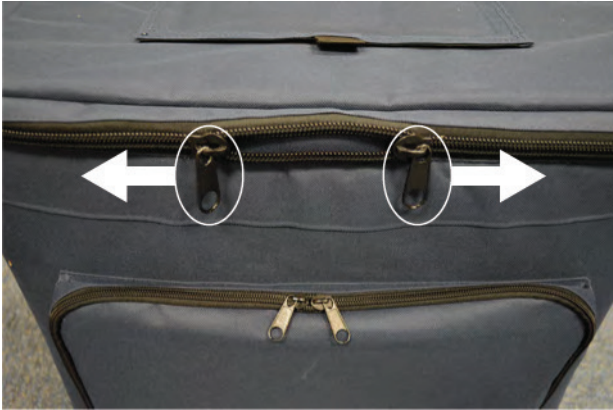

**Step 2:** Peel back the velcro-attached inner liner.



**Step 3:** Find loose rigid panel at rear of tote.



**Step 4:** Push rigid panel firmly down into bottom of Rolling Tote, filling out the corners.



**Step 5:** Reattach velcro of inner liner.



Your **Therapak**<sup>®</sup> Rolling Tote is ready to use!

