

Intac UA System

Urinalysis Transport System

Instructions for use

Required Components:

- 1 x beaker style collection cup with pour spout
- 1 x Intac UA Transport System



Step 1

Void urine into beaker collection cup.



Step 2

Remove cap from urinalysis transport tube and transfer urine from beaker collection cup into urinalysis tube. Please note the minimum fill line on the tube label.



Step 3

To eliminate the possibility of leakage, replace the cap securely by pushing down onto the cap until both surfaces meet.



Therapak's Intac Urinalysis transport system is designed to maintain the viability of urine specimen for up to 96 hours without refrigeration. Ideal for use in clinical trial site collection and transport to a designated testing laboratory.

For a copy of preservative tablet studies, please contact the Therapak Quality Assurance Department at 626-357-5900.