

Aqui-Pak[™]

Absorbs, Separates and Cushions

Aqui-Pak Absorbent Pouch and Tube Separator Ideal for Diagnostic Specimen Shipments



Aqui-Pak 4 bay pouch #10314

- Integral part of meeting new DOT/IATA Regulations
- Place between primary and secondary container
- Insert up to six (6) tubes in one Aqui-Pak
- Absorbs up to 150ml of liquid specimen
- Roll it up and place it in a bag or canister

Aqui-Pak was designed for the safe transport of laboratory specimen tubes. Aqui-Pak consists of a multi-layered polymeric material that has extremely high absorption properties. It is very convenient when used in combination with a specimen bag or canister to hold up to six tubes

Aqui-Pak is an integral solution for packaging Diagnostic Specimens according to the recently published DOT Hazardous Materials Regulations (HMRs) and IATA Dangerous Goods Regulations. Both the DOT and IATA have established packaging requirements for Diagnostic Specimens. Among other instructions for preparing blood and urine tubes (primary containers) inside of transport bags, canisters (secondary watertight receptacles) and boxes (rigid outer container), blood and urine tubes considered Diagnostic Specimens must be packaged with some form of absorbent material that not only separates multiple primary containers to prevent breakage, but cushion them within the packaging.

According to the regulations, absorbent material must be placed between the primary and secondary container and must be capable of absorbing the fluid content of all primary containers within the secondary packaging. Aqui-Pak absorbs, separates and cushions which eliminates the need for multiple product selection for packaging and transporting Diagnostic Specimens. Aqui-Pak is the cost efficient solution for compliance.



Aqui-Pak 6 bay pouch #10316

ORDERING INFORMATION:

Therapak No.	Description	Packed
10310	Aqui-Pak Single bay absorbent pouch	3,000/case
10335	Aqui-Pak 2 bay absorbent pouch	4,000/case
10314	Aqui-Pak 4 bay absorbent pouch	900/case
10316	Aqui-Pak 6 bay absorbent pouch	900/case

TO ORDER CONTACT:

Therapak, LLC
651 Wharton Drive
Claremont, California 91711

Ph: (909) 267-2000
Fax: (909) 267-2001
www.therapak.com

For sales information, call
(888) 505-7377