




Using the 31325 Protective Polybag for Corrugated Plastic Shipper

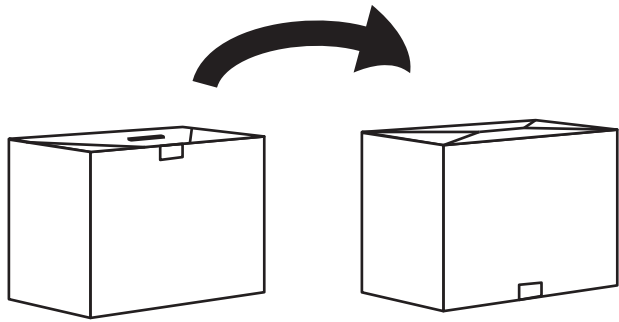


Step 1
Before filling, invert the corrugated shipper.

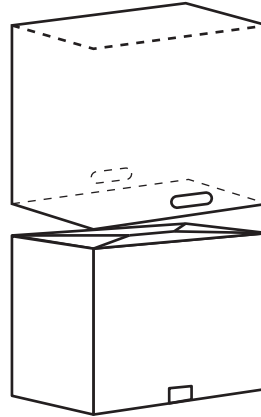


Step 2
Unfold the Protective Polybag and slide over shipper bottom.

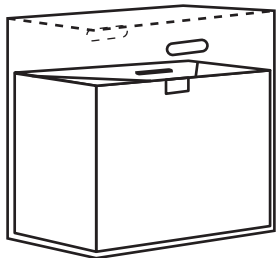


Step 3
With bag on shipper bottom, turn entire assembly back over. Pull bag up completely.

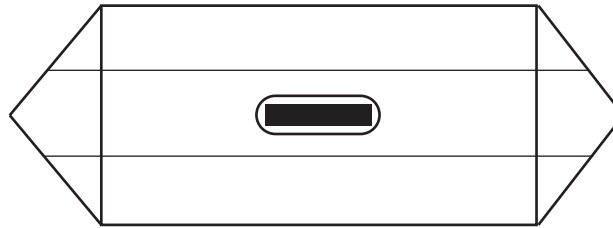

(top view)
Step 4
Fold top of bag so holes align with handle and excess plastic folds over sides of shipper.

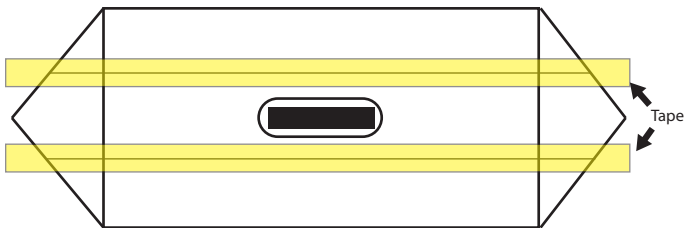

(top view)
Step 5
Apply packing tape across top of shipper, sealing bag flap and securing excess plastic to shipper sides.

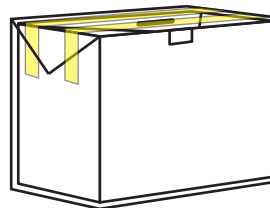

Step 6
Apply required labels to outside of Protective Polybag and ship.

31325 out of the box



31325 on the Corrugated Plastic Shipper

